

MediMatrix DICOM Viewer

Requirements:

Windows based computer/laptop, Internet access, logged into computer as Admin user.

Purpose:

The DICOM Viewer is an complete functioning DICOM based viewer for viewing DICOM images. This viewer has NOT been approved for diagnostic use by the FDA. By downloading and using the viewer you are abiding by this fact and other licensing agreements and assume all responsibilities for its use.

Steps:

1. Uninstall any current versions:
 - Start->All Programs WebInterstate DICOM Viewer
 - Un-Install DICOM Viewer Application
2. Installing the MediMatrix Eye Dicom viewer
 - Save the file to your local computer, then click it to run the install.
 - Double click WebiViewer_windows_x_x_x.exe
 - Follow the prompts - accept the license agreement and use the default settings
3. Configuration and Use
 - See “Viewer Features” pdf

For questions or assistance, please contact:
support@medimatrix.com
888-346-3030 x1

