

Modality Gateway Download

Requirements:

Windows based computer/laptop, Internet access, logged into computer as Admin user.

Purpose:

The Modality Worklist is a small software component that is used to retrieve patient demographics and order information from MediMatrix so the technologist does not have to type it in during image capture.

Steps:

1. Uninstall any current versions:

-Start->All Programs->WebInterstate MediMatrix Modality Worklist Un-Install Modality Service Application

2. Installing the MediMatrix Modality Gateway

-Save the file to your local computer, and then click it to run the install.

-Double click mwlgateway_windows_x_x_x.exe

-Follow the prompts - accept the license agreement and use the default settings

3. Configuring Modality Gateway Service

-Start->All Programs->WebInterstateMediMatrix Worklist Gateway->Modality Worklist UI

-TasksConfigure (click drop down box at bottom of screen and click edit)

Logging:

-Change the 'mwl.log.file.level' field to ALL if you are having problems.

-Change the 'mwl.log.verbose' field to TRUE if you are having problems.

Modality Worklist Settings:

-Usually you can set the IP to local host (127.0.0.1) and port 6104.

-Select the appropriate time zone locale.

Email Settings:

-Put your email address in the 'mwl.mail.from.address' field

-Put support@medimatrix.com in the 'mwl.mail.to.address' field.

Instructions continued on page 2



Modality Gateway Download

Web Access Settings:

- Create a user in the User Manager with GATEWAY SERVICE rights.
- Log into the Provider Application using this user so you can create a PIN.
- Use this user's information along with your MediMatrix secure domain (ie. <https://medimatrix.webinterstate.com>)
- Tasks Services (click stop service, then start service)
- *You will have to restart the service anytime you make configuration changes

4. Configure your Capture/DICOM Software program to retrieve information from the Modality Worklist using the values for AE Called Title, IP Address, and Port in the configuration settings.

For questions or assistance, please contact:

support@medimatrix.com

888-346-3030 x1

