

MediMatrix DICOM Viewer

Purpose: Import, view, save or export DICOM images.

Requirements: Windows based computer.

Import an Image(s):

- Open MediMatrix Eye Dicom Viewer
- File Open Search for Dicom Images

Saving an Image(s):

- File Save Image
- Select image(s) you want to save
- Choose location to save on machine
- Give the image/file a name (.dcm) for DICOM format
- Image height and width will default to how it was created by the modality
- Select file type and save

DICOM Export Set-Up:

- File DICOM Export Setup
- Remote AE Title is the name/title of your PAC system
- Local AE Title will rarely be used and may be same as Remote AE field
- Enter IP Address of PAC system
- Enter Port Address of PAC system
- Save Changes
- Verify DICOM connection to see if connected to PACs machine

Exporting ONE Image: (DICOM export setup must be completed)

- Double click image from image tree on left for exporting
- File Export Image
- You will then receive that same message stating image was transferred

Exporting ALL Images: (DICOM export setup must be completed)

- File Export ALL
- You will receive a message stating all images were transferred

For questions or assistance, please contact:

support@medimatrix.com

888-346-3030 x1

