

Storage Gateway Download

Requirements:

Windows based computer/laptop, Internet access, logged into computer as Admin user.

Purpose:

The Storage Gateway is a small software component that is used to route images for archiving into a local PACS.

Steps:

1. Uninstall any current versions:

-Start->All Programs->WebInterstate MediMatrix Storage Worlist Un-Install Storage Service Application

2. Installing the Storage Gateway

-Save the file to your local computer, and then click it to run the install.

- Double click Storagegateway_windows_x_x_x.exe

-Follow the prompts - accept the license agreement and use the default settings

3. Configuring Storage Gateway Service

- Start->All Programs->WebInterstate->MediMatrix Storage Gateway->Storage Gateway UI

-TasksConfigure (click drop down box at bottom of screen and click edit)

Application Logs:

-Change the 'store.log.file.level' field to ALL if you are having problems.

Success Logs:

-Change the 'store.success.log.backlog' field to 10 or more.

Storage Server Settings:

-'store.server.polling' can be set to 3,6,9,12,15 minutes. 12 minute is standard.

-'store.server.timezone' - pick your time zone.
SCU Setup

Below sets up the communication with your local PACS server

-'store.pacs.calledtitle' - the local PACS call application title

-'store.pacs.ipaddress' - the PACS ip address on your local network

-'store.pacs.storeas' - Store as the original transfer syntax, store in little endian uncompressed transfer syntax

-'store.pacs.port' - the port address

Email Settings:

-Use your email address in the 'store.mail.from.address' field

-Use support@medimatrix.com in the 'store.mail.to.address' field.

-Use your email address in the 'store.daily.report.mailto.address' - error and problem notifications are sent to this address

Instructions continued on page 2



Storage Gateway Download

Web Access Settings:

- Create a user in the User Manager with GATEWAY SERVICE rights.
- Log into the Provider Application using this user so you can create a PIN.
- Use this user's information along with your MediMatrix secure domain (ie. <https://medimatrix.webinterstate.com>)
- Tasks Services (click stop service, then start service)
- *You will have to restart the service anytime you make configuration changes

4. Configure settings on the PACS if needed.

For questions or assistance, please contact:
support@medimatrix.com
888-346-3030 x1

